



Compatible with any  
2, 4 or 6 Point  
Spreader Bar

# PATIENT SPECIFIC SLINGS

## REPOSITIONING, STRETCHER, MORTUARY SLING

Can Be Used For Bathing & Showers, Pools



Also available with  
"Key-Hole"  
Opening Clips

### GENERAL INFORMATION

Your Patient Specific (Disposable) Sling is a High Quality Product with many safety features incorporated within the material design and content of the product.

Single Layer with the Webbing Straps running across the width of the Sling.

Double Layer with the Webbing Straps attached on the outer perimeter of the Sling.

Each Sling is size/colour co-ordinated on the Binding, Handles, Displacement Straps.

Single Layer Sling is approved for a load of 500 Kg (or 1100 Lbs)

The Patient Specific Slings are compatible with most hoists (lifters) Spreader Bars.

The Patient Specific Slings can also be used for Bathing and Showers and also in Pools.

Use with confidence - the Slings are not going to rip or tear or fall apart! Dispose of sling after bathing, etc and replace with a new sling.

The Patient Specific Slings can also be left behind under the Patient. The Slings are 100% breathable.

The Patient Specific Slings are intended to be used as a Patient Specific Product and should not be recycled and re-distributed.

### SLING COMPATIBILITY

The Sling with webbing Attachments are compatible for use with any 2,4,OR 6 Point spreader Bar Lift System.

### Wash Indicator


The Patient Specific Slings incorporate a Wash Indicator Label. In the event of the sling being washed (and then recycled for use again) the Wash Indicator will have changed design so that the Care Provider is aware that the Sling has been washed.

**Do not re-use and dispose of the Sling.**

#### Prior to Washing

**WASH**

I  
N  
D  
I  
C  
A  
T  
O  
R

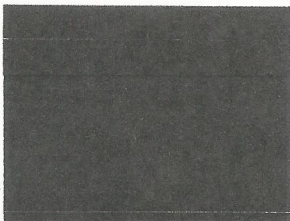


The Wash Indicator will high light that the Sling has been washed. Do Not Use the Sling!



#### After Washing

**WASH**

I  
N  
D  
I  
C  
A  
T  
O  
R

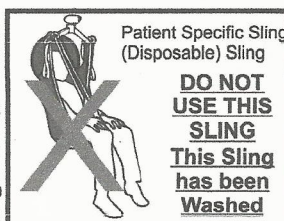

FOR MORE INFORMATION OR INQUIRIES PLEASE CONTACT YOUR LOCAL REPRESENTATIVE



Barrier Free Lifts  
1620 SW 17TH STREET  
OCALA FL34471  
1-800-582-8732  
USA

# THE REPOSITIONING, STRETCHER, MORTUARY SLING

## The Repositioning, Stretcher, Mortuary Sling

The Sling is designed for use with most patients and is available in one size to Fit All and because it is 100% breathable, can be left behind and under the patient.


Use the Sling with Confidence in Bathing, Showers and Pools.

As a Repositioning Sling - The Sling can be used to move the patient up and down and can also be used as an aid in turning the patient.


As a Stretcher Sling - The Sling can be used as a "Total Sling" in various applications.

As a Mortuary Sling - The Sling can be used as a "Transport Sling" for use with Cadavers; as a "Cover" and as a "Left-in Place" in Cadaver Storage and on Mortuary and Autopsy Tables.

**PLEASE NOTE: THE SLING MAY ONLY BE FITTED FROM A LYING POSITION**

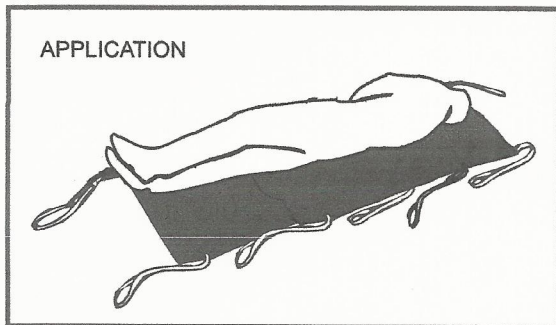


Draw sheet roll the patient onto the sling.  
Straighten and smooth out the sling.

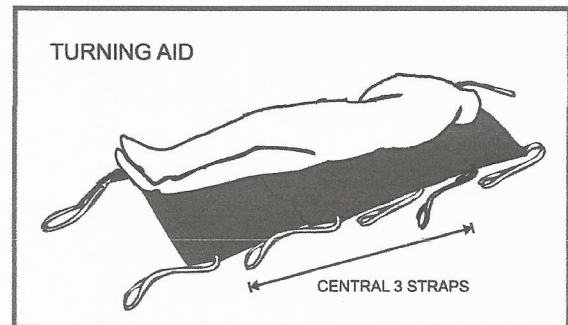

The Sling can be used as an aid in Turning the Patient.

Use any combination of the 3 Central Straps.

Attach any 3 of the Central Straps to the Lifter. Lift the sling up and the Patient should roll to the opposite side of the attachment - see drawing above.

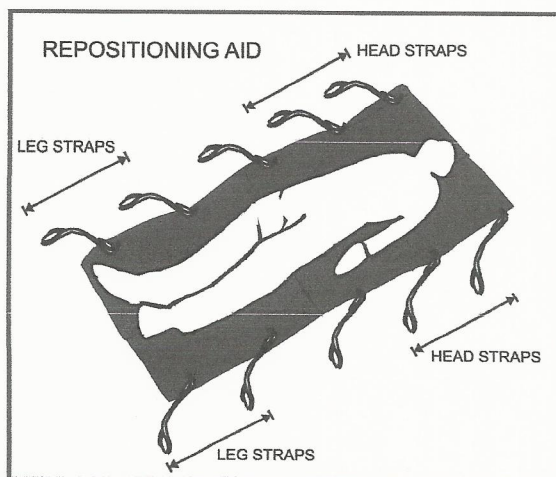

The Sling can also be used as a Repositioning Aid.

To move the patient up the bed, use any combination of the Head Straps. Ensure that both the straps on either side of the sling are used.

To move the patient down the bed, use any combination of the Leg Straps. Ensure that both straps on either side of the sling are used.

Attach the Straps to the lifter and pull up and along, making sure that the position of the lifter is in line with the desired position.