



Compatible with any  
2, 4 or 6 Point  
Spreader Bar

## PATIENT SPECIFIC SLINGS Standard and Fullback

Can Be Used For Bathing & Showers, Pools



Also available with  
"Key-Hole"  
Opening Clips

### GENERAL INFORMATION

Your Patient Specific (Disposable) Sling is a High Quality Product with many safety features incorporated within the material design and content of the product. The Slings are available in Standard Style (Slings without Head Support) - 2 sizes, and Fullback Style ( Slings WITH Head Support) in 5 sizes.

Each Sling is size/colour co-ordinated on the Binding, Handles, Displacement Straps.

The Slings are approved for a load of 300 Kg (660 lbs) or 500 Kg (1100 lbs) for the Extra Extra Large Size Sling. The Patient Specific Slings are compatible with most hoists (lifters) Spreader Bars.

The Patient Specific Slings can also be used for Bathing and Showers and also in Pools. Use with confidence - the Slings are not going to rip or tear or fall apart! Dispose of sling after bathing, etc and replace with a new sling.

The Patient Specific Slings can also be left behind under the Patient. The Slings are 100% breathable.

The Patient Specific Slings are intended to be used as a Patient Specific Product and should not be recycled and re-distributed.

**Dispose of Sling after use.**

### SLING COMPATIBILITY

The Slings with Webbing Attachments are compatible for use with any 2, 4 or 6 Point Spreader Bar Lifter System.


### Wash Indicator

The Patient Specific Slings incorporate a Wash Indicator Label. In the event of the sling being washed (and then recycled for use again) the Wash Indicator will have changed design so that the Care Provider is aware that the Sling has been washed. **Do not re-use and dispose of the Sling.**

#### Prior to Washing

##### WASH

I  
N  
D  
I  
C  
A  
T  
O  
R

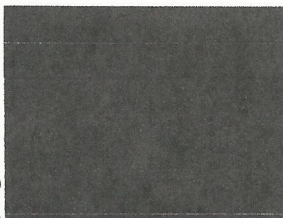

The Wash Indicator will high light that the Sling has been washed. Do Not Use the Sling!



#### After Washing

##### WASH

I  
N  
D  
I  
C  
A  
T  
O  
R




FOR MORE INFORMATION OR INQUIRIES PLEASE CONTACT YOUR LOCAL REPRESENTATIVE




Barrier Free Lifts  
1620 SW 17TH STREET  
OCALA FL34471  
1-800-582-8732  
USA

## PATIENT SPECIFIC (DISPOSABLE) APPLICATION INSTRUCTIONS

### FULLBACK AND STANDARD SLINGS - FITTING FROM A SEATED POSITION

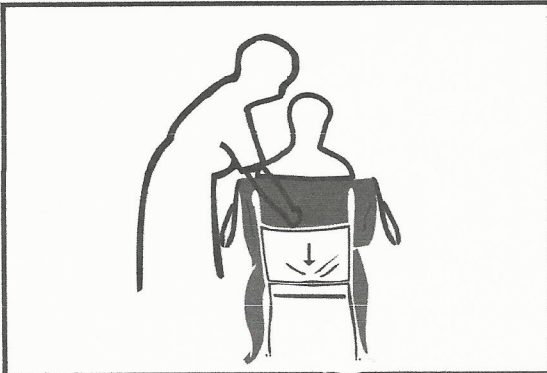


Apply and push the Sling down - as above. Ensure that the base of the Sling is about an inch further down and below the base of the spine.

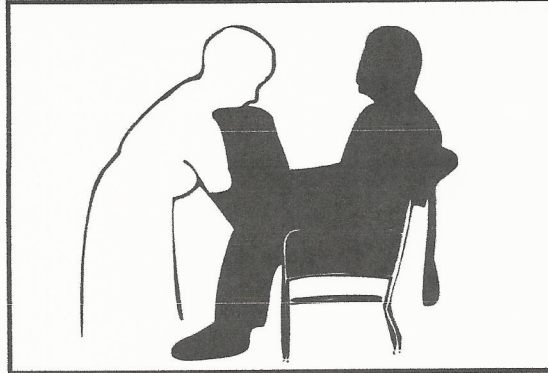


Pull the legs of the sling through, under the thighs, and at a right angle with the long bone of the thighs. Check that the legs are not twisted.

### FULLBACK AND STANDARD SLINGS - FITTING FROM A SEATED POSITION (Continued)

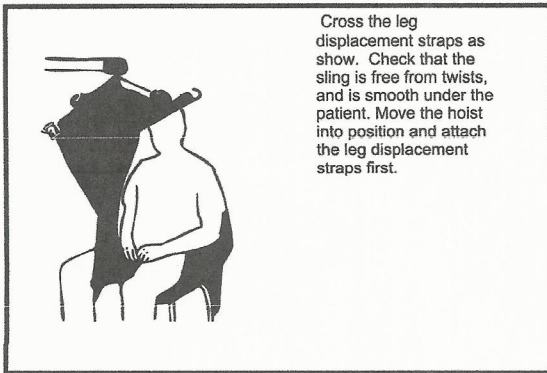


Cross the leg displacement straps as show. Check that the sling is free from twists, and is smooth under the patient. Move the hoist into position and attach the leg displacement straps first.

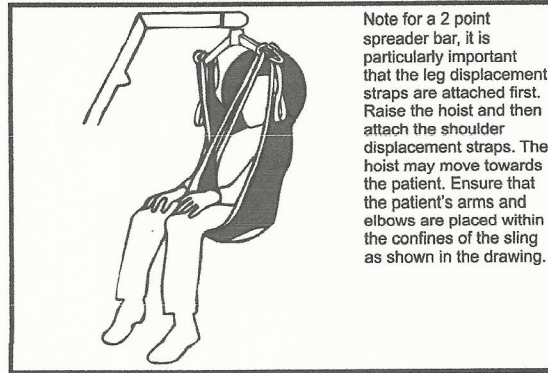


Note for a 2 point spreader bar, it is particularly important that the leg displacement straps are attached first. Raise the hoist and then attach the shoulder displacement straps. The hoist may move towards the patient. Ensure that the patient's arms and elbows are placed within the confines of the sling as shown in the drawing.

### FULLBACK AND STANDARD SLINGS - ALTERNATIVE SEATED POSITION

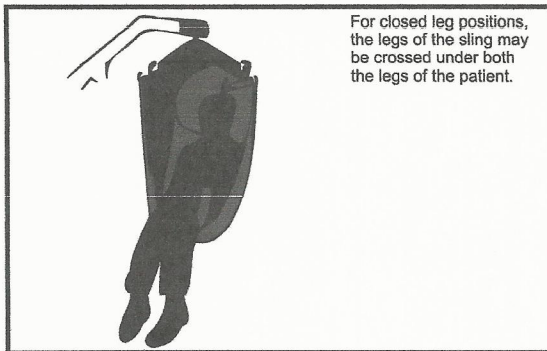

For closed leg positions, the legs of the sling may be crossed under both the legs of the patient.

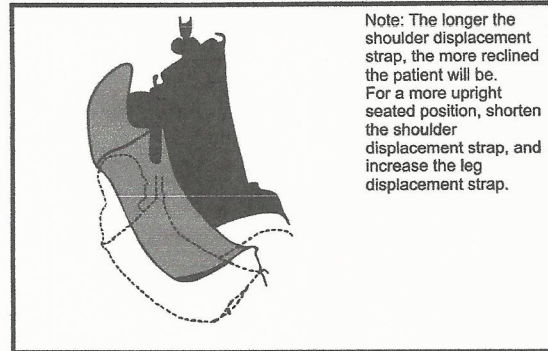

Note: The longer the shoulder displacement strap, the more reclined the patient will be. For a more upright seated position, shorten the shoulder displacement strap, and increase the leg displacement strap.

Size	Approx. Patient Weight	Binding Color	Safe Working Load	Coccyx to Top of Head	Width at Hips
XXS	0-55 lbs / 0-25 kg	ORANGE	660 lbs / 300 kg	20-21 in / 51-53 cm	under 9 in / 23 cm
XS	56-75 lbs / 25-34 kg	BLUE	660 lbs / 300 kg	21-24 in / 54-60 cm	9-12 in / 23-30 cm
S	75-124 lbs / 34-56 kg	RED	660 lbs / 300 kg	24-28 in / 60-70 cm	13-15 in / 33-38 cm
M	125-174 lbs / 57-79 kg	YELLOW	660 lbs / 300 kg	28-33 in / 70-84 cm	16-18 in / 41-46 cm
L	175-249 lbs / 79-113 kg	GREEN	660 lbs / 300 kg	33-38 in / 84-96 cm	19-22 in / 48-56 cm
XL	250-399 lbs / 113-181 kg	PURPLE	660 lbs / 300 kg	38-43 in / 96-108 cm	22-26 in / 56-66 cm
XXL	400-600 lbs / 181-272 kg	WHITE	1100 lbs / 500 kg	43-47 in / 108-120 cm	over 26 in / 66 cm
XXXL	600-1100 lbs / 272-499 kg	GREY	1100 lbs / 500 kg		